


COVID-19

Emergency Response Quick Setup Guide

This V+Pro Emergency device has been pre-configured with initial settings to begin treatment of adults suffering from the symptoms associated with COVID-19.

This guide provides basic setup instructions for VOCSN use in an emergency response situation. In-depth instructions are available online at [VentecLife.com/COVID-19support](https://www.ventec.com/COVID-19support).

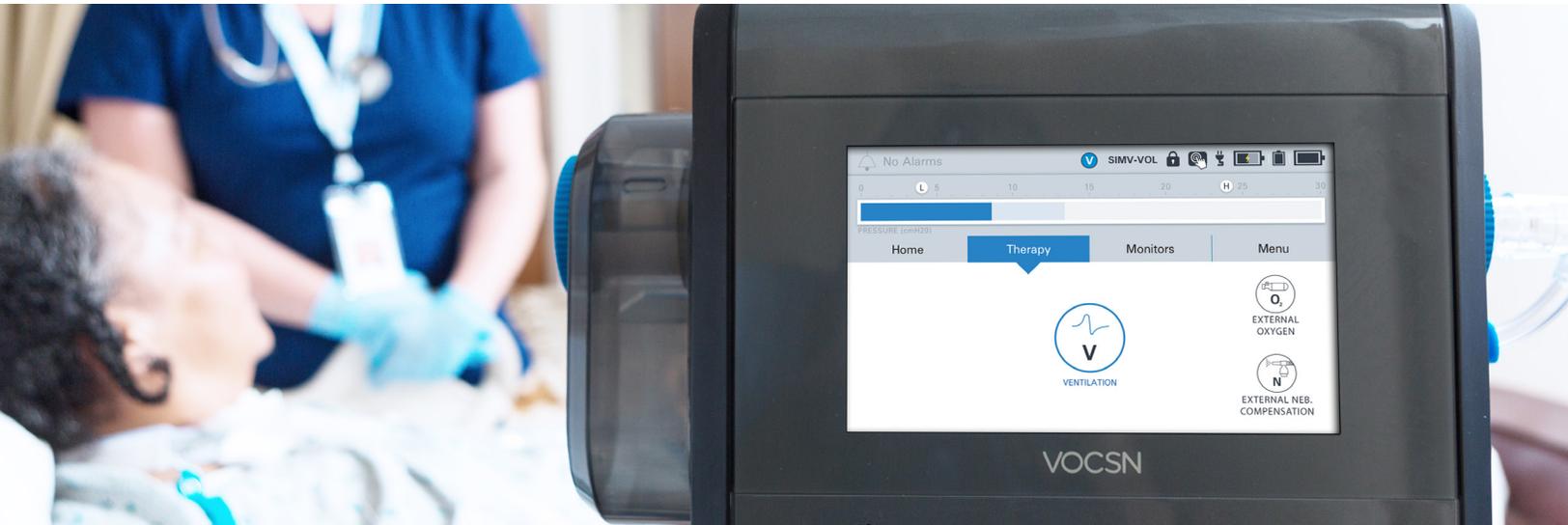

1 CONNECT FILTER AND PATIENT CIRCUIT

There's an "Active" circuit type in the box.


VOCSN



VOCSN
BACTERIAL
FILTER
(in box)



PATIENT CIRCUIT
"Active" circuit type in box shown.

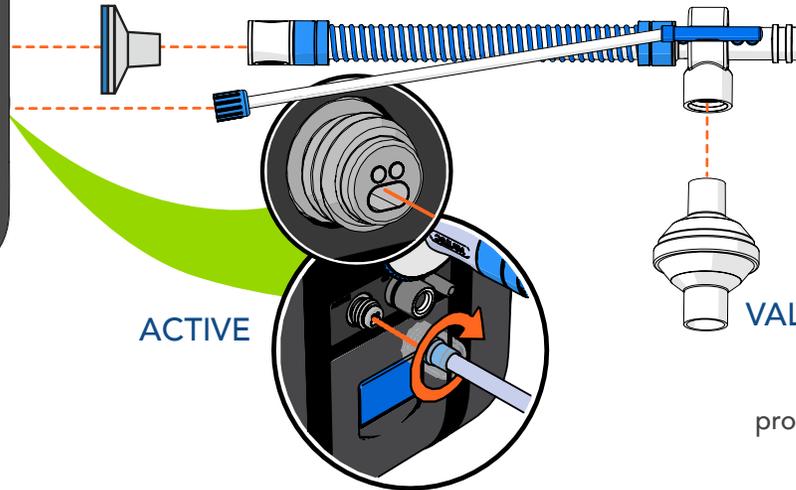

CONNECT
PATIENT
LATER
(STEP 8)


ACTIVE

VALVE BACTERIAL
FILTER

(not in box:
provided by hospital)


2 CONNECT EXTERNAL OXYGEN

Connect high-pressure or low-pressure oxygen. **After** connecting it, **turn the oxygen source on**.

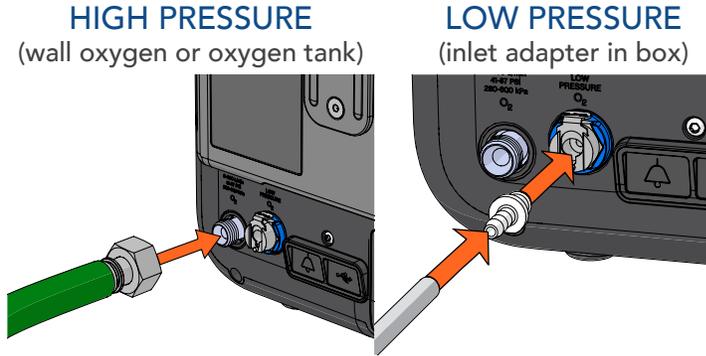

3 CONNECT POWER

Plug the power cord into an outlet and the power block. Then, plug the other end into the back of VOCSN and screw the connector clockwise.

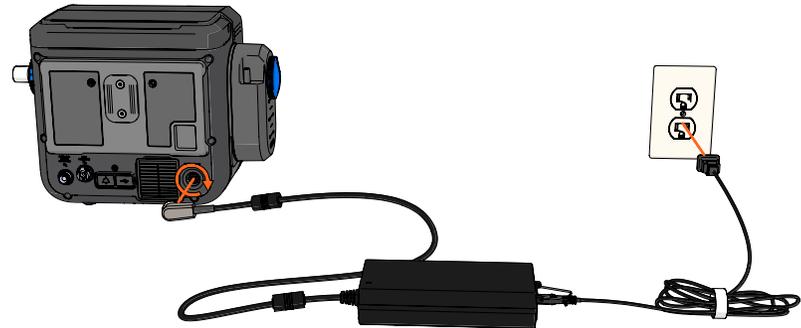

4 POWER ON

Press the On/Off button on the lower right side of VOCSN.

5

RUN A PRE-USE TEST

Carefully **follow the on-screen instructions** that appear after power-up. You will be asked to perform actions with the patient circuit.

After the Pre-Use Test, ventilation will begin automatically. **Wait** to connect the patient until step 8.

TOUCHSCREEN

ALARM SILENCE

ON/OFF





6 VERIFY VENTILATION SETTINGS

V+Pro Emergency has four main navigation tabs - **Home**, **Therapy**, **Monitors**, and **Menu**, and three therapy buttons - **Ventilation**, **External Oxygen**, and **External Neb. Compensation**

FOUR MAIN NAVIGATION TABS

THERAPY TAB CONTROLS

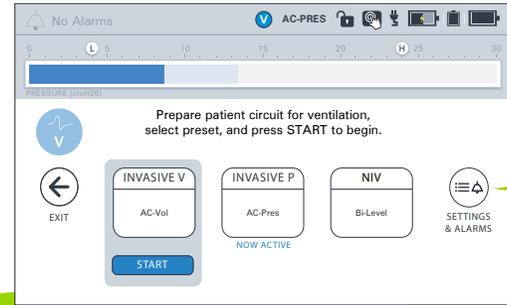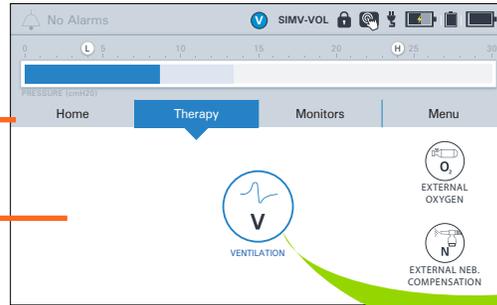

VENTILATION SCREEN



SETTINGS & ALARMS

Verify Ventilation therapy settings are appropriate for the patient.


VOCSN will automatically deliver volume control ventilation for an average adult patient with an endotracheal tube (preset **INVASIVE V** is active by default). To switch to pressure control ventilation, press **INVASIVE P** to highlight it, and then press the **START** button beneath it.

To view or change Ventilation settings, highlight the preset you would like to view or modify, press **SETTINGS & ALARMS**, and then scroll to find the setting (such as **PEEP** or **Tidal Volume**). To change a setting, press **EDIT**.

7 START OXYGEN

Press the **Therapy** tab and then the **External Oxygen** button to open oxygen controls.
You must press START to activate oxygen.

- **HIGH** preset delivers 100% FiO2 from an external high-pressure oxygen source
- **MEDIUM** preset delivers 60% FiO2 from an external high-pressure oxygen source
- **EXT LOW P** preset delivers bleed-in oxygen from a low-pressure oxygen source

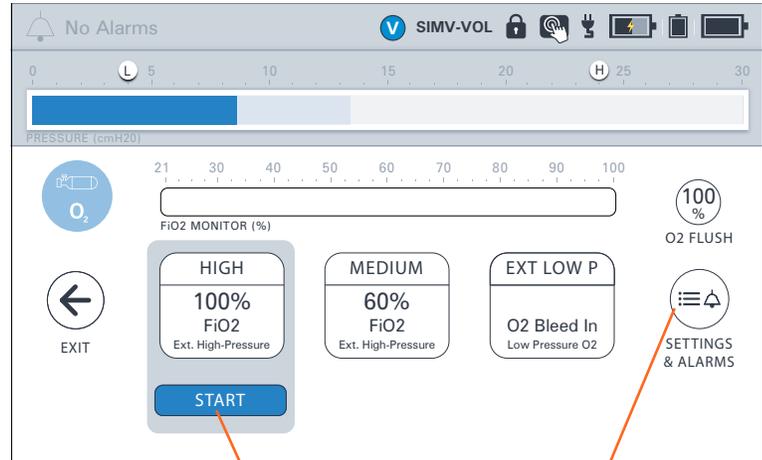


Press **START**
to begin oxygen

If needed, press
SETTINGS & ALARMS to
change settings

8 CONNECT PATIENT

Connect the patient circuit to a flex tube, HME(F), and then the patient's endotracheal tube. VOCSN is now providing Ventilation and Oxygen to the patient.

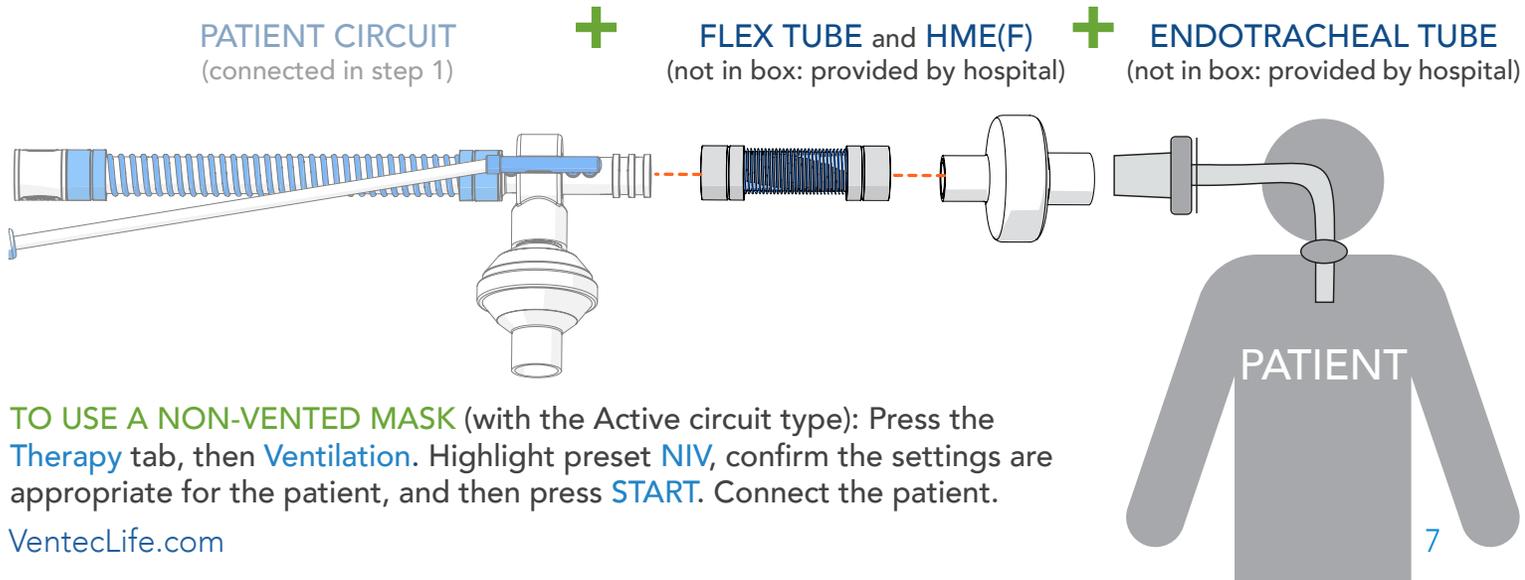

TO USE A NON-VENTED MASK (with the Active circuit type): Press the **Therapy** tab, then **Ventilation**. Highlight preset **NIV**, confirm the settings are appropriate for the patient, and then press **START**. Connect the patient.

Additional Resources

For manuals, training videos, and additional clinical resources, see our COVID-19 support page online at:

VentecLife.com/COVID-19support

